

## Congratulations on the purchase of you new AVID MiniTracker 4

### Basic Instructions

1. Remove your new scanner from packaging.
2. Ensure batteries are installed correctly.
3. Press the Power button.
4. Read your first Microchip by holding down the read button- the number will be displayed once read!

### Need Bluetooth?

1. Press \* button to go to the Settings.
2. Scroll down to Bluetooth and switch it on.
3. On your PC or handheld device search for the scanner which will be called AVIDxxxxxx and then pair.
4. That's it – Enjoy this your new scanner.



For complete user guide  
Scan the QR code above or  
visit [avidid.com](http://avidid.com)

Read Button



Power Button

